








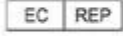
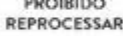

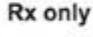
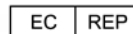


SYMBOL	DESCRIÇÃO	DESCRIPTION	DESCRIPCIÓN
	PRODUTO ESTERILIZADO POR RADIAÇÃO GAMA	PRODUCT STERILIZED THROUGH GAMMA RAYS	PRODUCTO ESTERILIZADO POR RADIACIÓN GAMA
	NÃO REUTILIZAR	DO NOT REUSE	NO LO REUTILICE
	CONSULTAR AS INSTRUÇÕES DE USO	CONSULT INSTRUCTIONS FOR USE	CONSULTE LAS INSTRUCCIONES DE USO
	MARCAÇÃO CE	CE MARK	MARCA CE
	MANTENHA SECO	KEEP DRY	MANTÉNGALO SECO
	MANTENHA AO ABRIGO DO SOL	KEEP AWAY FROM SUNLIGHT	MANTÉNGALO LEJOS DE LA LUZ SOLAR
	NÃO UTILIZAR SE A EMBALAGEM ESTIVER VIOLADA	DO NOT USE IF PACKAGE IS DAMAGED	NO LO UTILICE SI EL ENVOLTORIO ESTÁ DAÑADO
	NÃO REESTERILIZE	DO NOT RESTERILIZE	NO LO REESTERILIZAR
	ATENÇÃO	CAUTION	PRECAUCIÓN
	REPRESENTANTE AUTORIZADO NA COMUNIDADE EUROPEIA	AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY	REPRESENTANTE AUTORIZADO EN LA COMUNIDAD EUROPEA
	PROIBIDO REPROCESSAR	PROHIBITED REPROCESS	PROHIBIDO REPROCESSAR
	LIMITE SUPERIOR DE TEMPERATURA	UPPER LIMIT OF TEMPERATURE	LÍMITE SUPERIOR DE TEMPERATURA
	ATENÇÃO: A LEI FEDERAL (EUA) LIMITA A VENDA DESTES DISPOSITIVOS POR OU POR ORDEM DE UM PROFISSIONAL DE SAÚDE LICENCIADO.	CAUTION: FEDERAL LAW (USA) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTHCARE PRACTITIONER.	PRECAUCIÓN: LAS LEYES FEDERALES (USA) RESTRINGEN LA VENTA DE ESTE DISPOSITIVO POR O EN EL ORDEN DE UN PROFESIONAL DE LA SALUD LICENCIADO.



## DEVELOPED AND MANUFACTURED BY:

### S.I.N. Sistema de Implante Nacional S/A

CNPJ [Corporate Taxpayer's Registry]: 04.298.106/0001-74  
Rua Soldado Ocimar Guimarães da Silva, 2445 - Vila Rio Branco CEP: 03348-060 - São Paulo - SP - Brazil  
Phone/Fax: +55 (11) 2169-3000

## SERVICE TO PROFESSIONALS

0800 770 8290  
+55 (11) 2169-3000  
[www.sinimplante.com.br](http://www.sinimplante.com.br)  
e-mail: [sin@sinimplante.com.br](mailto:sin@sinimplante.com.br)

## OVER-REV-UNIPessoal LDA.

Rua General Ferreira Martins, nº10 - 8D 1495-137  
Algés - Portugal


## RESPONSIBLE TECHNICIAN:

Alessio Di Rasio  
CREA-SP: 5061207169

## PRODUCT:

S.I.N. CAD-CAM Transfer  
**Anvisa Registration:** 80108910076

**S.I.N. CAD-CAM TRANSFER is intended for expert procedures, which must be performed by qualified professionals. The use of the product and surgical techniques are inherent to the professional's training. The use of the product must be performed in a surgical environment and in proper conditions for the health and safety of the patient.**

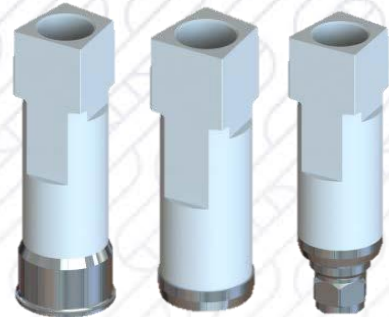

## PRODUCT DESCRIPTION

The S.I.N. CAD-CAM Transfer consists of a conical or square abutment following the seating platforms according to each implant or abutment model, supplied together with a fixation screw. They are available to professionals in STERILE form by Gamma Radiation.

### Information applicable to JBHE 34; JBHE 41; JBHE 45; JBHE 50 models:

- Product with pass screw (with screw);
- Manual screwing torque on the implant;
- Compatible with External Hexagon implants.

### Information applicable to JBHI 38 ; JBHI 45; JBHI 50 models:

- Product with pass screw (with screw);
- Manual screwing torque on the implant;
- Compatible with Internal Hexagon implants.

### Information applicable to JBSWCM model:

- Product with pass screw (with screw);
- Manual screwing torque on the implant;
- Compatible with Cone Morse implants.

### Information applicable to JBUKM model:

- Product with pass screw (with screw);
- Manual screwing torque on the implant;

- Compatible with UNITITE Cone Morse implants.

### Information applicable to JBMA model:

- Product with pass screw (with screw);
- Manual screwing torque on the component;
- Compatible with mini-abutment components.

### Information applicable to JBMMA model:

- Product with pass screw (with screw);
- Manual screwing torque on the component;
- Compatible with Micro Mini-Abutment components.

### Information to models: JBAC 00; JBAC 06:

- Product with pass screw (with screw);
- Manual screwing torque on the component;
- Compatible with Conic Abutment components.

## INDICATIONS OF USE

The S.I.N. CAD-CAM Transfer is indicated to copy and transfer the position of the implant to a virtual work model where the prosthesis will be made.

## OPERATION PRINCIPLE

Its working principle is based on the transfer of the implant or installed abutment in the oral cavity through an intraoral scan or an extraoral model.

## HOW TO USE

- Stage 01:** Definition of the implant or abutment feature;
- Stage 02:** Definition of the CAD-CAM transfer to be used;
- Stage 03:** Screwing of the transfer over the implant or abutment;
- Stage 04:** Scanning of the intraoral transferor or in the model;
- Stage 05:** Finalization of the prosthesis on the Analogue installed in the model or directly in the mouth;
- Stage 06:** Fixation of the prosthesis through screw.

## CONTRAINDICATIONS

As long as the material is used properly, there is no contraindication of use. Transient use product used only to copy and transfer the position of the implant.

## SIDE EFFECTS

If the technique is not adequate and the patient is not subjected to the indicated tests, the final result of application of the S.I.N. CAD-CAM Transfer may not succeed, generating product loss or fracture.

## PRECAUTIONS AND RECOMENDATIONS

For the placement of S.I.N. CAD-CAM Transfer, it is recommended that the professional has a specialization course in the area and prepare a prosthetic execution plan.

## WARNING

**SINGLE USE PRODUCT – STERILE**  
Sterilization Process: Gamma Radiation.  
Non-ensured sterilization if the seal is broken.

## TRACEABILITY

All S.I.N. - Sistema de Implante products have sequential batches that allow traceability, which promotes greater safety for the professional qualified to the procedure. Through this batch number, it is possible to know the entire history of the product from the manufacturing process to the distribution time.

## STORAGE

S.I.N. CAD-CAM TRANSFER must be stored in a cool and dry place, keep away from sunlight.

## TRANSPORTATION

S.I.N. CAD-CAM TRANSFER must be transported in room temperature, away from direct sunlight, avoiding places where large variations of temperature and humidity occur. The transportation must be carried out properly to avoid falls and it must be carried out in its original package.

## HANDLING CONDITIONS

The S.I.N. CAD-CAM TRANSFER is a sterile product that should be handled only in a sterile field by properly trained professionals and in appropriate scrubs at the time of the surgical procedure.

## ADDITIONAL INFORMATION

Single use product. Prohibited Reprocessing.  
Exclusive dental use product.

## DISPOSAL OF MATERIAL

The disposal of materials should comply with local hospital regulations and applicable local laws.

## EXPIRATION DATE

Indicated on the label.